Venturi Muffler Instructions For Use


- **INTENDED USE:** The Venturi Muffler is intended for use in conjunction with the MaxVenturi air/oxygen mixer by connecting onto the air inlet filter. This device decreases noise generated during the normal operation of the MaxVenturi.

⚠ **WARNING:** This device is not a filter and must be used in conjunction with the manufacturer's recommended inlet filter.

⚠ **WARNING:** This device does not contain any user serviceable or replaceable components. Do not attempt to clean the internal surfaces or muffler element of this device for any reason. If any form of contamination is suspected, discard immediately.

INSTALLATION: Firmly connect to inlet of filter as shown in graphic.

- CLEANING:** The outside surfaces of this device may be cleaned using isopropyl alcohol or a mild soap solution. Use caution to ensure no cleaning agents enter inside the device as this may result in harmful vapors being delivered to a patient's airway. Do not attempt to sterilize device. If for any reason it is suspected that the interior of the device may be contaminated, discard the device immediately and replace with a new unit. Cleaning should be performed between patient uses or as needed.

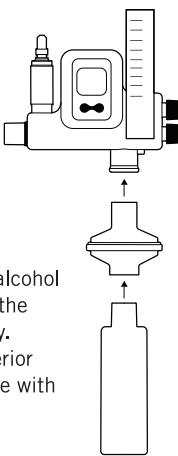Authorized Representative:
QNET BV
Kantstraat 19
NL-5076 NP Haaren
The Netherlands

Maxtec I 2305 South 1070 West
Salt Lake City, Utah 84119 I USA
TEL: 385.549.8000 I FAX: 801.973.6090
www.maxtec.com

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